

Welcome

Your Gastric Alimetry Test

Date: _____

Time: _____

Duration of Test: _____

Location: _____

In preparation for your test:

- Do not eat or drink for 6 hours before the test
- Follow your doctor's advice on stopping medications
- Ensure you do not have any rashes, sunburn, wounds, abrasions, infected or inflamed areas on your abdomen.
- Do not wear makeup or moisturiser on your abdomen
- Remove any piercings on your abdominal area

As part of your Gastric Alimetry assessment, Alimetry will collect and process your data, including personal information, to generate the Gastric Alimetry Report based on your results. Alimetry partners with Galen Data Inc., a medical-grade cloud provider, to host the Alimetry Cloud server. Information collected by clinicians using the Gastric Alimetry device and application may be stored in the Alimetry Cloud servers as part of the service provided. Alimetry adheres to all relevant privacy and data protection laws. For further information, please contact privacy@alimetry.com. If you have any questions or concerns, please reach out to your healthcare provider.

PM-002 Ver.2 November 2024 © Alimetry Limited
Alimetry and Gastric Alimetry are trademarks of Alimetry Limited




What is Gastric Alimetry?

Gastric Alimetry is a noninvasive test designed to help diagnose stomach problems. The test involves placing a sticky patch of sensors and a recording device onto the skin overlaying the stomach.

The Gastric Alimetry device is able to read the activity of the stomach through a technology called Body Surface Gastric Mapping. The data allows doctors to see patterns that may relate to various symptoms.

During the test, you will record your symptoms into an App, which gives doctors a clearer understanding of your symptom experience.

There is an optional wellbeing survey included in the test which allows your clinician to gain a more comprehensive understanding of your experience.

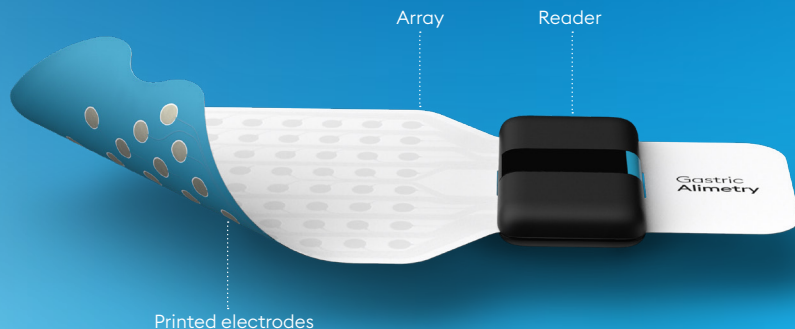

How Does It Work?

1. Wear the Device

The abdomen is prepped, and an adhesive electrode array is placed on your abdomen to collect the signals of your stomach. A reader is attached to the array which records these signals.

2. Record Your Symptoms

Every 15 minutes during the test you will be prompted to update your current symptoms. The types of symptoms you can record during the test include:

- Bloating
- Belching
- Nausea
- Vomiting
- Reflux
- Heartburn
- Stomach burn
- Upper gut pain
- Excessive fullness
- Feeling full too quickly

3. Eat a Meal

After a 30 minute baseline recording period, you will eat a light meal to stimulate stomach activity.

You will sit in a comfortable reclined position for the duration of the test. You will be able to get up to use the toilet or take a comfort break if necessary. The entire test takes around 5 hours to set up and run.

You are able to read or use an electronic device during the test. However, you will be asked to limit your movement and to avoid talking or sleeping during the test.

Are There Any Risks?

There may be some mild redness of the skin due to exfoliation. Some patients may also notice a mild itch or rash.

What Happens After?

After the test is complete, a report is generated from the data collected during the test. Your clinician will interpret it and share the results with you.

